*No. 1 choice
of hospitals
& mothers*

Harmony™

- EN** Instructions for use
- ES** Manual de instrucciones
- PT** Manual de instruções
- EL** Οδηγίες χρήσης
- HE** הוראות שימוש

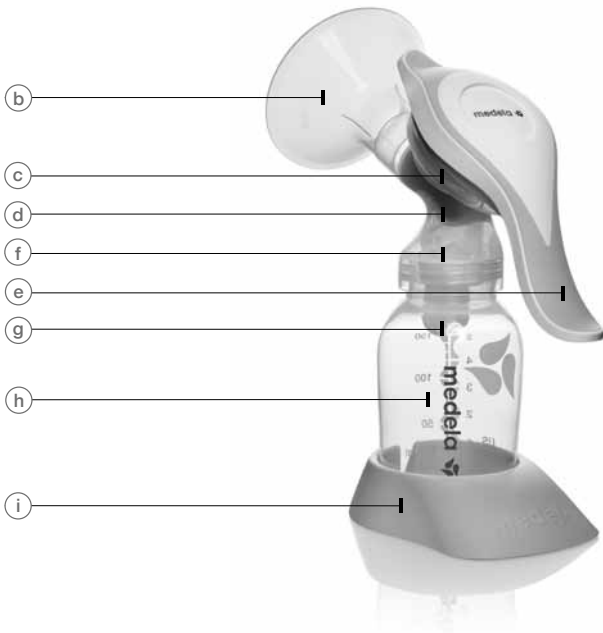
EN Parts List XI

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
I Harmony™ Breastpump

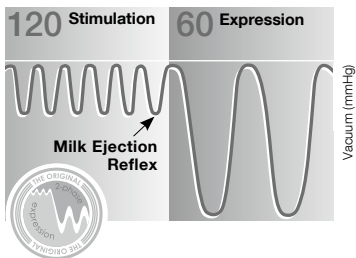Mother's milk is the best nutrition for babies. Harmony gives you and your baby optimal support during the breastfeeding period. The attractive manual breastpump is especially suitable for occasional pumping at home or on the go.

Harmony has **2-Phase Expression®**, the only research based breastpump technology to mimic a baby's natural nursing rhythm.

Stimulation:

Fast pumping rhythm to start your milk flowing.

Expression:

Slower pumping rhythm to express your milk gently and efficiently.

Proven to feel most like your baby and to reduce your pumping time.

II Indications For Use

The Harmony Breastpump is a manual breastpump:

- To be used by lactating women to express and collect milk from their breast during absences from the baby (e.g. an evening out from baby, working part-time).
- To relieve engorgement occasionally.
- For home use and on the go.

III Important Safeguards

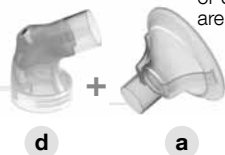
Read all instructions before use.

- Use the product only for its intended use as described in this instruction manual.
- **Use Medela original accessories only.**
- Check pump kit components for wear or damage.
- This is a single user product. Use by more than one person may present a health risk.
- If problems or pain occur, consult your lactation consultant or physician.

SAVE THIS INSTRUCTION MANUAL

IV Assembly Instructions

1

Make sure breastshield (PersonalFit or SoftFit Breastshield) and connector are securely attached.

2

Snap membrane onto valve until membrane lies completely flat.

IV Assembly Instructions

3

Push assembled valve and membrane onto connector. Ensure sidewise position of valve as indicated on picture.

f g + d a

4

Screw bottle into connector.

a d f g

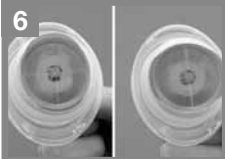
h

5

If the handle components have been disassembled, place the oval shaped base of the stem into the diaphragm.


6


Make sure the oval shape is lined up correctly in the diaphragm.

Correct

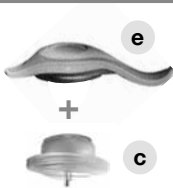Incorrect

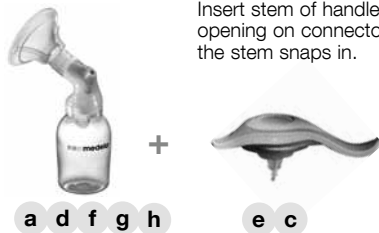Place diaphragm and stem into pump handle and press down firmly around entire edge to ensure proper seal.

Insert stem of handle assembly into opening on connector. Make sure that the stem snaps in.

V Pumping

Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.

Center your breastshield over your nipple.

Do not tilt or overfill bottles when pumping.

V Pumping

2

Start pumping with **Stimulation** by pressing down on the milk drops symbol on top of the handle. Press and release quickly to imitate a baby's fast suckling until your milk starts to flow or you sense milk ejection.

3

Once your milk starts to flow or you sense milk ejection, change to **Expression** by pressing the lower end of the handle. This will give you a longer stroke that imitates a baby's slower, deeper sucks. The level of vacuum is controlled by the compression and the speed of the handle stroke.

Once finished pumping disassemble the pump kit items. Close the bottle and store it for later use.

Use bottle stand to prevent tip over.

VI Breastshield Fitting

Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

Medela PersonalFit™ Breastshield: Available in five sizes – because one size does not fit all.

VII Cleaning and Hygiene

Prior to first and after each use

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after each use to prevent milk residues from drying up and to avoid bacterial growth!
- Disassemble all parts prior to cleaning. Make sure to remove the yellow valve head (part no. 810.0475) from the connector and separate the white valve membrane (part no. 820.7013) from the valve head.
- Hand wash and rinse all disassembled parts of your pump kit that come into contact with the breast and breast milk (i.e. parts b+c+e+f+g+k+l+m) in hot, soapy water using a detergent till all visual residues are removed.
- Air-dry parts or dry with a clean towel.
- Use damp (not wet) cloth to wipe the motor unit or handle after each use.
- Optionally, boil disassembled parts for 5 minutes in a saucepan filled with water (with an optional tea spoon of citric acid) or steam - or microwave - sterilize.

Tubing care

Inspect the tubing after each pumping session. If condensation appears or if milk accidentally gets into the tubing turn off the pump and unplug from power source. Remove the tubing and clean it according to the instructions «Prior to first and after each use». Shake water droplets out and hang up to air-dry.

Note: It is not recommended to use Steam - or Microwave - sterilizing for tubing.

Hygiene

Note: Never use creams, oils or lotions when breastpumping. This can cause damage to the breastshield and may contaminate the breastmilk.

- For the treatment of dry/sore nipples we recommend the use of Purelan.
- If creams, oils or lotions are used, clean the breast and surrounding area before every pumping session.
- Do not use soap or alcohol which can dry the skin and cause cracked nipples.
- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Avoid touching the inside of bottles and lids.
- Store parts in clean zip lock bag or store in a container with lid or alternatively wrap parts in a clean paper or cloth towel.
- Replace breastshield if it shows any sign of wear.

VII Cleaning and Hygiene

Please follow these handling and cleaning instructions.
Medela can assume no liability for a wrong handling.

	Accessories			Breastshields		Breastpumps	
	Bottles	Valves & Membranes	Cap, Lid, Disc, Stand	PersonalFit	SoftFit	Handle Harmony	Connector Harmony
Handwash, Dishwasher, Boil	✓	✓	✓	✓	✓	✓	✓
Microwave Bags	✓	✓	✓	✓	✓	✓	✓
Steam Sterilizer	✓	✓	✓	✓	✓	no	✓
Autoclave	no	✓	✓	✓	no	no	no

VIII Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.
- Label bottles/bags with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).
- Gently shake the bottle/bag to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

IX Troubleshooting

If there is low or no suction

Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See «Assembly Instructions».

X Disposal

The unit is made of plastics and may only be disposed of in accordance with local regulations.

XI Parts List

Article numbers

a	SoftFit™ Breastshield (24 mm)	810.0402
b	PersonalFit™ Breastshield (24 mm)	810.7084
c	Diaphragm/stem with O-ring	800.0766
d	2-Component Connector	800.7196
e	Handle	800.7170
f	Valve head	810.0475
g	Valve membrane	820.7013
h	Bottle (150 ml)	811.0021
i	Bottle stand	810.0462
j	Cap	810.0474
k	Lid	810.7062
l	Disc	810.0473

*Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.
Subject to change.
Content may vary from picture.*

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Medela AG, Medical Technology